

PERSONAL REUSABLE THERMOMETER

Personal Contact

Fire Department

Hospital

Doctor

Police

911

Emergency

NexTemp®
Personal Clinical Fever
Thermometer



Last black dot
shows 98.6°F

To Read:



Last black dot
shows 37.3°C



MADE IN USA

Directions: Oral Use


Place dot end of the thermometer far back under the tongue like a conventional glass thermometer. Wait at least **ONE MINUTE** before removing.

Remove thermometer and read immediately.

Directions: Axillary Use

Place thermometer high in the axilla (armpit) with the dot-matrix against the torso and parallel to the length of the body. Close arm to hold thermometer in place. Wait at least **3 MINUTES** before removing.

Remove thermometer and read immediately.

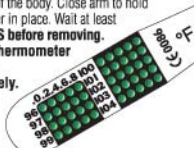

To Read:

The last dot to turn **BLACK** indicates the correct temperature.

The thermometer will reset itself after about 30 seconds. Normal oral and axillary temperatures are between 97°F and 99°F.

To Clean and Reuse:

Wipe thermometer with alcohol-swab or wash with hot soapy water, rinse, wipe dry and store in this handy case. This thermometer should only be used by one person to prevent cross-infection.

This product is protected under patents #5,678,465, #6,257,759, #6,284,078 and #6,491,774.

NAME:

Lot: **03590** Expires: **DEC 2009** #70-5107-01