


Home Care Nursing Reusable Oral Thermometer



NexTemp
Personal Clinical Fever
Thermometer

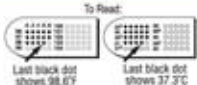

To Read:

CE
0086

MADE IN USA

Directions: Oral Use

Place dot end of the thermometer far back under the tongue like a conventional glass thermometer. Wait at least **ONE MINUTE** before removing.

Remove thermometer and read immediately.

Directions: Axillary Use

Place thermometer high in the axilla (armpit) with the dot-matrix against the torso and parallel to the length of the body. Close arm to hold thermometer in place. Wait at least **3 MINUTES** before removing.

Remove thermometer and read immediately.

To Read:
The last dot to turn **BLACK** indicates the correct temperature. The thermometer will reset itself after about 30 seconds. Normal oral and axillary temperatures are between 97°F and 99°F.

To Clean and Reuse:
Wipe thermometer with alcohol-swab or wash with hot soapy water, rinse, wipe dry and store in this handy case. This thermometer should only be used by one person to prevent cross-infection.

This product is protected under patents #5,678,465, #6,257,759, #6,284,078 and #6,491,774.

NAME:

Lot: 03592 Expires: DEC 2009 #70-5155-01