

NAME:

Lot: 03571 Expires: DEC 2009 #70-5095-04

This product is protected under patents #5,676,465, #6,257,759, #6,284,078 and #6,491,774.

Wipe thermometer with alcohol-swab or wash with hot soapy water, rinse, wipe dry and store in this handy case. This thermometer should only be used by one person to prevent cross-infection.

To Clean and Reuse:

The thermometer will reset itself after about 30 seconds. Normal oral and axillary temperatures are between 97°F and 99°F.

To Read:

The last dot to turn BLACK indicates the correct temperature.

Place dot end of the thermometer far back under the tongue like a conventional glass thermometer. Wait at least **ONE MINUTE** before removing.

Remove thermometer and read immediately.**Directions: Axillary Use**

Place thermometer high in the axilla (armpit) with the dot-matrix against the torso and parallel to the length of the body. Close arm to hold thermometer in place. Wait at least

3 MINUTES before removing.

Remove the thermometer and read immediately.

Directions: Oral Use

Place dot end of the thermometer far back under the tongue like a conventional glass thermometer. Wait at least **ONE MINUTE** before removing.

Thermometer

Personal Clinical Fever

NextTemp®



MADE IN USA



Last black dot shows 98.6°F

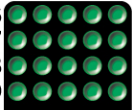
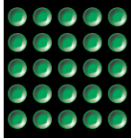


Last black dot shows 97.3°F

To Read:

Environmentally Friendly

.0.2.4.6.8 100

96
97
98
99101
102
103
104

Reusable

Easy to Read

Latex-Free

Mercury-Free Replacement Thermometer

Your message here

NextTemp®