

# Directions for Axillary Use

*NexTemp<sup>®</sup>, NexTemp<sup>®</sup> Ultra and Tempa•DOT<sup>®</sup>*


MEDICAL  
INDICATORS




Open the package, fold along perforation, and tear off a single thermometer strip. Peel back the top of the wrapper and remove the thermometer by pulling on the exposed handle.



Insert the thermometer under the patient's arm, high in the axilla. Be sure to place the dot matrix towards the patient's body, so it's touching the torso, not the inside of the arm.

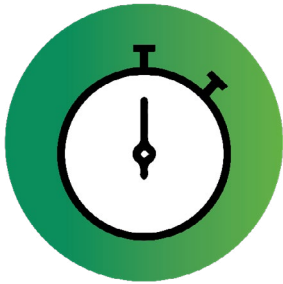

Have the patient close their arm to keep the thermometer in place and wait 3 minutes.

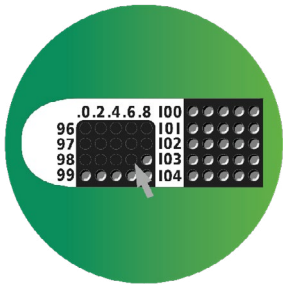

Remove the thermometer, wait approximately 10 seconds, and then read the temperature. The last blue dot on the matrix indicates the correct temperature for Tempa•DOT<sup>®</sup> thermometers, and the last black dot indicates the correct temperature for NexTemp<sup>®</sup> and NexTemp<sup>®</sup> Ultra thermometers.



The temperature will remain visible for 20 seconds with NexTemp<sup>®</sup>, a full 1-minute with NexTemp<sup>®</sup> Ultra, and the temperature reading will permanently lock in with Tempa•DOT<sup>®</sup>. Once you have read and recorded the temperature, discard the thermometer.