

Directions for Rectal Use

NexTemp® Plus and Tempa•DOT® Plus


MEDICAL
INDICATORS




Open the package and remove one thermometer from the dispenser on the top of the box. Do not remove the plastic sheath.



Lubricate the thermometer and insert into the rectum until the dot matrix and numbers are completely covered.

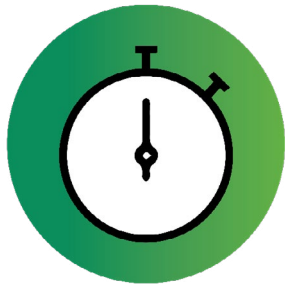

Wait 3 minutes before withdrawing the thermometer and sheath from the rectum (discard the sheath immediately).

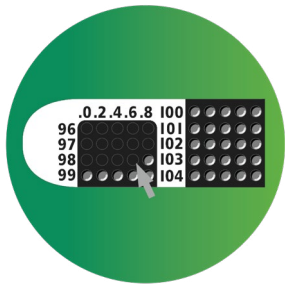

Wait approximately 10 seconds, and then read the temperature. The last blue dot on the matrix indicates the correct temperature for Tempa•DOT® Plus thermometers, and the last black dot indicates the correct temperature for NexTemp® Plus thermometers.



The temperature will remain visible for 20 seconds with NexTemp® Plus, and will permanently lock in with Tempa•DOT® Plus thermometers. Once you have read and recorded the temperature, discard the thermometer.