

TRAXIT® DIRECTIONS FOR USE




Open the package, fold along the perforation, and tear off a single thermometer strip.



Peel one thermometer from the backing and place it on the side of the patient's body, high in the axilla.



Have the patient close their arm, and wait three minutes before taking the first temperature reading.

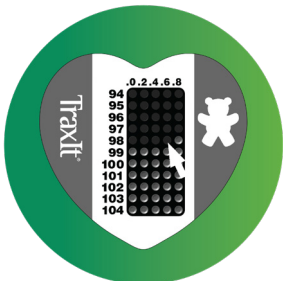

The last black dot on the matrix indicates the correct temperature. After the initial reading, all future readings are available instantaneously for a continuous 48-hours.



Once you are finished using and/or after the 48-hours is complete, remove the thermometer from the patient's underarm and discard.