


NEXTEMP PRO: DIRECTIONS FOR AXILLARY USE




Open the package, fold along perforation, and tear off a single thermometer strip. Peel back the top of the wrapper and remove the thermometer by pulling on the exposed handle.



Insert the thermometer under the patient's arm, high in the axilla. Be sure to place the dot matrix towards the patient's body, so it's touching the torso, not the inside of the arm.

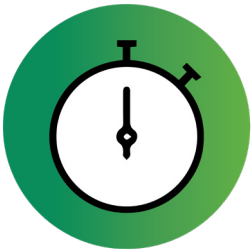

Have the patient close their arm to keep the thermometer in place and wait 3 minutes.

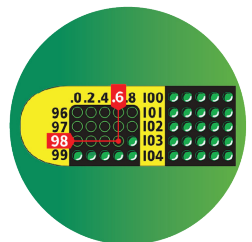

Remove the thermometer and then read the temperature. The last black dot indicates the correct temperature for NexTemp® Pro thermometers. The temperature will remain visible for 1-minute.



The NexTemp Pro thermometer can be used on the same patient, up to 30 times.



Once you have finished using, discard the thermometer in the regular trash.