


NEXTEMP PRO: DIRECTIONS FOR ORAL USE




Open the package, fold along perforation, and tear off a single thermometer strip. Peel back the top of the wrapper and remove the thermometer by pulling on the exposed handle.



Insert the thermometer into the patient's mouth, and place under the tongue as far back as possible into either side heat pocket (as you would any oral thermometer). The dots can face either up or down. Reminder: be sure to wait at least 15 minutes after eating, drinking or smoking.

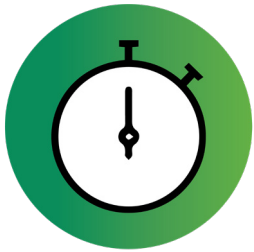

Have the patient press their tongue down on the thermometer while keeping their mouth closed for 1 minute.

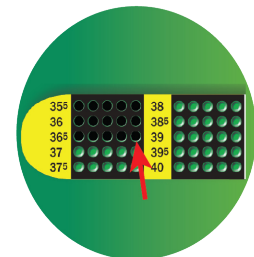

Remove the thermometer from the patient's mouth, and read the temperature. The last black dot indicates the correct temperature for NexTemp[®] Pro thermometers. The temperature will remain visible 1-minute.



The NexTemp Pro thermometer can be used on the same patient, up to 30 times.



Once you have finished using, discard the thermometer in the regular trash.