



Wearable Children's Underarm Thermometer

Directions:

Peel off both release liners from the back to remove thermometer and place thermometer in the Axilla (armpit)

Warning: Read Before Use!

To ensure TraxIt® performs at its best, it's essential to follow all usage instructions carefully:

- Make sure the underarm area is clean and dry before application. The skin must be free of oil, hair, and powder
- Place TraxIt® deep in the arm pit under the fold and press gently for good contact.
- Correct placement is key! TraxIt must be applied directly under the child's armpit-not lower on the chest
- Close the arm and wait 3 minutes. Readings will then continue automatically for up to 48 hours

Proper placement ensures accurate and consistent temperature tracking.

TraxIt can be used on patients 6 months and older as per clinical study conducted.

How to Read:

1. Lift arm to read. Thereafter readings are virtually instant. Just lift the arm to take the next measurement.
2. Look at dot grid. The highest black dot indicates the correct temperature (see example).

Note:

TraxIt® will adhere to the armpit through bathing and may be left in place up to 48 hours on normal, healthy, intact skin.

To Remove:

Gently peel off and discard. DO NOT REUSE.

Accuracy: $\pm 0.2^{\circ}\text{F}$ (maximum permissible error).

Thermometer has no temperature offset.

Do Not Reuse:

Reuse may result in compromising product integrity, lead to device failure, or may create risk of wound contamination, patient infection or cross infection which may lead to injury, illness or death of the patient.

Conforms to ASTM E 1299-96 Clinical thermometers Part 2:

Phase change type (dot matrix) thermometers.

Last black dot indicates 104.8°F



Do Not Reuse

Not made with natural rubber latex



Store below given temperature

MD: Medical Device