

INSTRUCTIONS FOR ORAL USE

NexTemp® Go

1 Remove thermometer from wrapper

2 Place under tongue as far back as possible.
Close mouth for 10 seconds.

3 Remove thermometer and read the temperature immediately.
The highest black dot indicates the correct temperature.

4 Record temperature and discard thermometer.



- Warning:
- Shorter/longer times can cause inaccurate temperature readings.
 - The green, yellow, and red color gradients are design elements only. Use numerical readings for discussions with your healthcare provider.
 - Do not use if unit package damaged or opened.

Conforms to ASTM E1299 with exclusion of section 5.3 and 5.6.
Accuracy $\pm 0.5^{\circ}\text{F}$ within range of 96.0°F to 105.5°F .



PAPIRÆS KE




MD



Do Not Reuse

Not made with natural rubber latex



30°C
86°F

Store below given temperature