


INSTRUCTIONS FOR USE

NexTemp® Ultra



1 Remove thermometer from wrapper

2 Oral use
Place under tongue as far back as possible. The dots can face either up or down. Reminder: be sure to wait at least 15 minutes after eating, drinking or smoking.

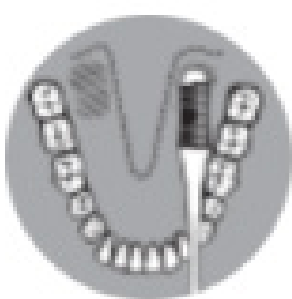

Axillary use
Please the thermometer under the patient’s arm, high in the axilla. Be sure to place the dot matrix towards the patient’s body, so it’s touching the torso, not the inside of the arm.



3 Oral use
Have the patient press their tongue down on the thermometer while keeping their mouth closed for 1 minute.

Axillary use
Have the patient close their arm to keep the thermometer in place and wait 3 minutes.

4 Remove the thermometer and read the temperature. The last black dot indicates the correct temperature. The temperature will remain visible for 1-minute

5 Record temperature and discard thermometer.

Accuracy: ±0.2°F within range of 96.0°F to 104.8°F.

Do Not Reuse: Reuse may result in compromising product integrity, lead to device failure, or may create risk of wound contamination, patient infection or cross infection which may lead to injury, illness or death of the patient.

Conforms to ASTM E 1299-96 and EN 12470-2:2000+A1:2009 Clinical thermometers Part 2: Phase change type (dot matrix) thermometers.



PAPIRÆS KE




MD



Do Not Reuse

Not made with natural rubber latex



30°C
86°F

Store below given temperature