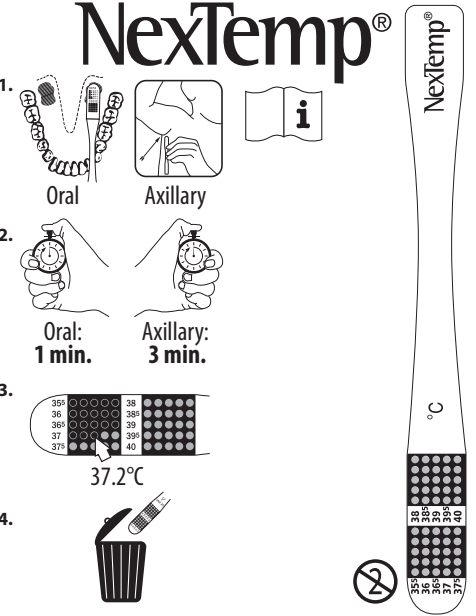
DA – Engangstermometer • Indhold: 100 termometre
Anvisninger – Peroral anvendelse:

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EN – Disposable Thermometer • Contents: 100 Thermometers
Directions— Oral Use:

- Hold along perforation and tear off a single thermometer. Peel back top of wrapper to expose handle and remove by pulling.
- Place in mouth under tongue, as far back as possible into either heat pocket, as you would any oral thermometer. Have patient press tongue down on thermometer while keeping mouth closed.
- Leave for a minimum of 1 minute – shorter times may give inaccurate readings.

Under Arm:

- Place under arm with green dots deep in axilla. Use arm to hold the thermometer in place for AT LEAST 3 MINUTES – shorter times may give inaccurate readings.

How to Read:

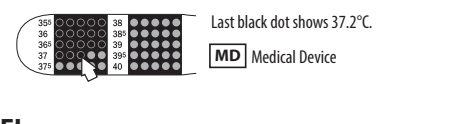
- Remove thermometer.
- Read temperature immediately.
- The highest black dot indicates the correct temperature (see example).
- Dispose of thermometer in accordance with local regulations.

Accuracy: +0.1°C to -0.2°C (maximum permissible error).
Thermometer has no temperature offset.

Do Not Reuse:

Reuse may result in compromising product integrity, lead to device failure, or may create risk of wound contamination, patient infection or cross infection which may lead to injury, illness or death of the patient.

Conforms to ASTM E 1299-96 and EN 12470-2:2000+A1:2009 Clinical thermometers Part 2: Phase change type (dot matrix) thermometers.

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©2026 Medical Indicators Inc.